VACO[®] by OPED **ped**

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Manual

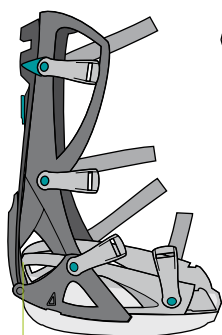
Manuel

Manuale
d'uso


Anlegevorgang
Application procedure
Procédure de mise en place
Procedura di applicazione


Ventil
Valve
Soupape
Valvola

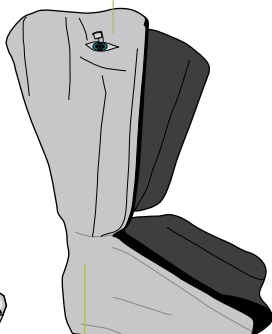Unterschale
Lower shell
Coque inférieure
Supporto
rigido inferiore

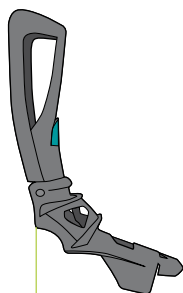Bezug
Fabric Cover
(beanbag inside)
Garniture
Rivestimento

Oberschale
Upper shell
Coque supérieure
Supporto
rigido superiore

1


VACOped anlegen | Applying VACOped Mettre en place VACOped | Applicazione di VACOped

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E Place your foot into the fabric inner boot. Close the fabric cover. With the inner boot on, step into the lower shell. Place the upper shell over the front of the fabric cover, fasten the straps.

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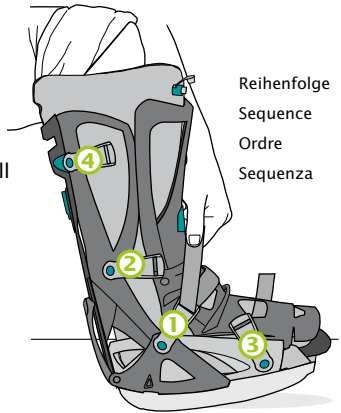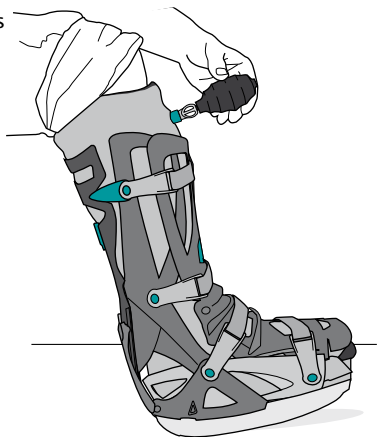
E Ensure the valve ring is in the "down" position (see valve function), attach gray end of pump to circular end of valve. Pump repeatedly until pump no longer inflates. Close the valve cap.

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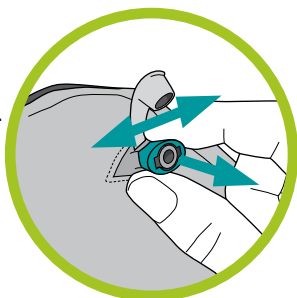
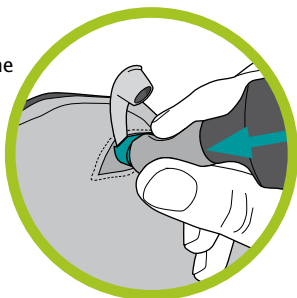
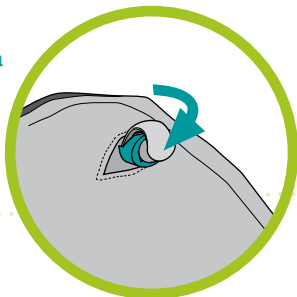
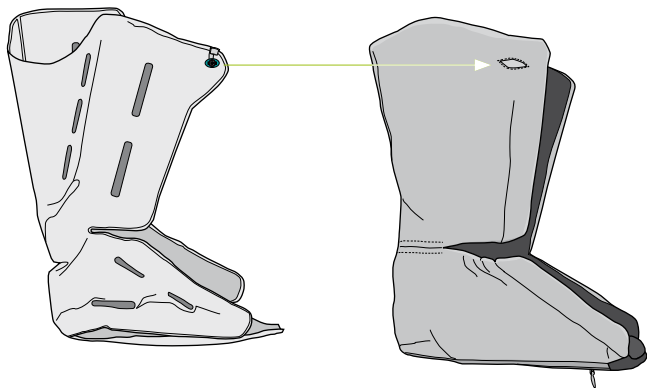
E Open the back zip completely. Remove the vacuum cushion. Make cushion soft by opening valve. Smooth out the vacuum cushion with your hand. Pump out air to make vacuum cushion hard. Put the vacuum cushion back into the new cover and insert the valve through the buttonhole

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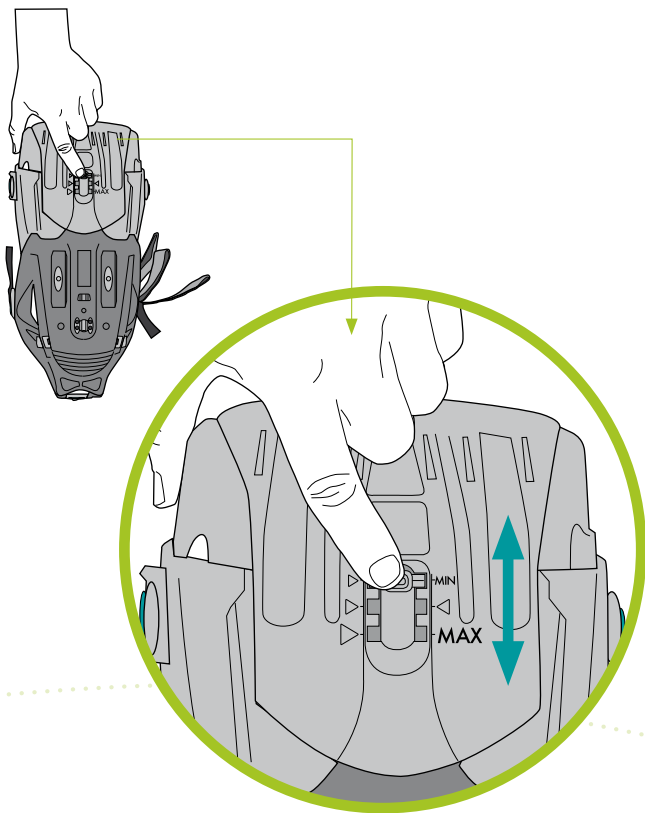
E Remove the sole (see removing the sole).
Press the release button.
Pull the toe piece to the required length.

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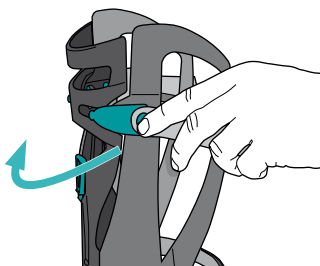
E Press the clasp to release it from the calf shell.

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
E Push the clasp into the desired position.

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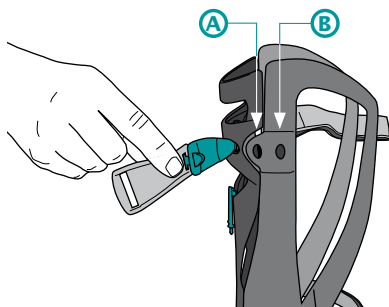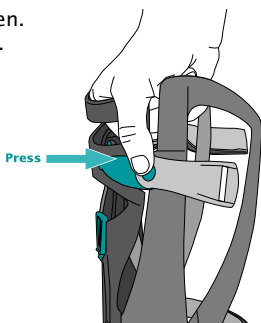
E Press the clasp into the desired hole. Repeat the procedure on the other side.

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Fixed ankle joint (immobilized) adjustment

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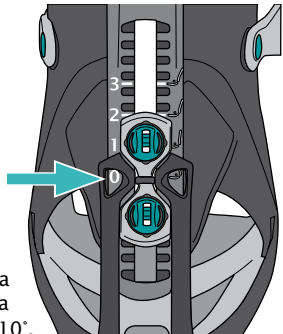
E Immobilization is possible in 5° increments from -15° up to +30°. Level “1” is equivalent to +10 degrees.

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Flexible ankle joint (ROM) adjustment

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E Top screw = sets plantar flexion limit (downward mobility of forefoot)

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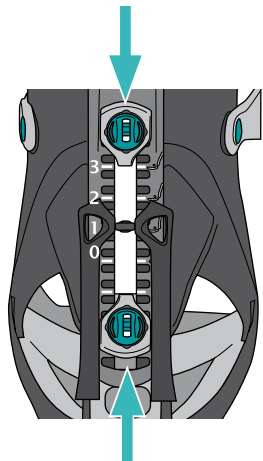
E Bottom screw = sets dorsal extension limit (upward mobility of forefoot)

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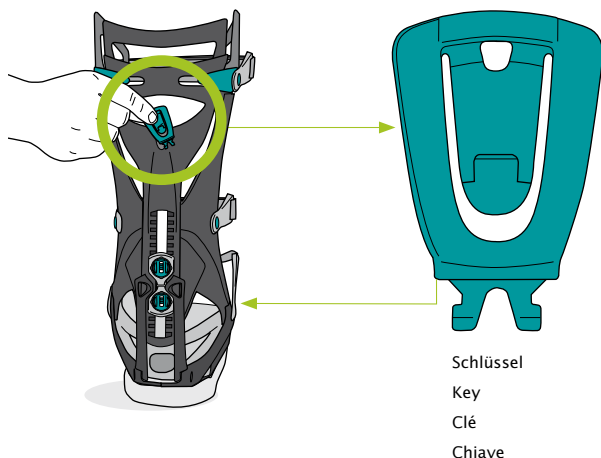
E Take the key out of its mounting and insert it in the screw.

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E Note: Please use the wedge sole for settings from 10° to 30°.

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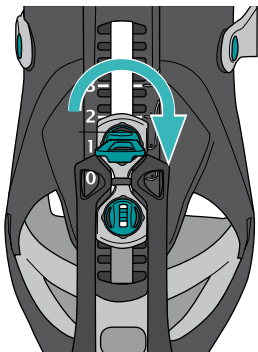
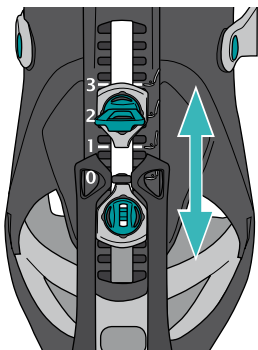
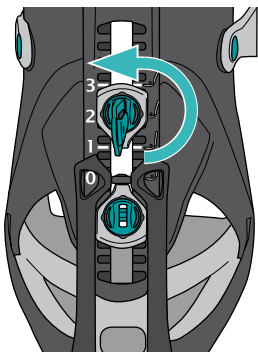
Useful Details

Cleaning and care

- Shell and straps: Shell dishwasher 90°C (194°F), straps washing machine 40°C (104°F).
 - Terry cloth covers: washing machine 40°C (104°F), close the zipper first.
 - Vacuum cushion: use disinfectant wipe or disinfectant spray.
 - Storage: store in a dry place.
-

Product features

- Can be worn on the right or left foot.
 - Sizes: Small 35–39 Medium 39–44 Large 44–50
-

Indications / types of use

- fractures of the foot and ankle joint
 - calcaneus fracture
 - Achilles tendon rupture
 - prosthetics
 - arthrodesis
 - serious ligament injuries
-

Troubleshooting

Tender spots

- Remove boot completely. Put air back in vacuum cushion. Shake cushion to redistribute beads. Re-apply boot.
- Place air back in vacuum cushion. Locate area on vacuum cushion that is source of tenderness, and use fingers to ensure beads are between tender spot and outer shell.

If repeated adjustment does not remedy the pain or numbness in the toes or feet and/or skin turns bluish, call your doctor immediately! Medical aftercare is necessary in this case! You will find numerous tips and tricks at www.vacoped.com

Safety notices / contraindications

- Check that the sole is properly attached to bottom of boot. If not, remove and re-apply
 - There are no known contraindications.
-

Technical data / parameters

Materials used

Hard shells (PP)*; covers (cotton/PU foam); cushion cover (PVC), padding material (EPS balls; top cover (PP/PU)*; straps (PP material)*; outer sole (EVA)*

Packaging

- Weight including product: 2,800g (6 lbs 2 oz)
 - Dimensions 58 x 37 x 18 cm (23" x 14.6" x 7")
-

Guarantee

The product guarantees comply with the statutory provisions.

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OPED GmbH
OPED AG
OPED Inc.