

# VACO<sup>®</sup>cast Pro/Achilles



Inspired by ski boots, **VACOcast Pro/Achilles** enables “active recovery” to get patients walking sooner without crutches.

## MEDICAL INDICATIONS

- Achilles tendon rupture
- Achilles tendinitis
- Ankle fractures
- Ankle joint replacement
- Arthrodeses of the upper and lower ankle joint
- Calcaneus fractures
- Complex ligament injuries
- Metatarsal fractures
- Severe sprains
- Soft tissue injuries with impaired healing process
- Splinting of foot and lower leg

## To order:

Contact Ken Dupuis  
OPED USA  
877.673.3426  
ken@opedusa.com

## Size Article Numbers

Small: VPED-01-4-S  
Medium: VPED-01-4-M  
Large: VPED-01-4-L

## Why vacuum?

Micro beads gently contours to the limb, as air gets sucked out of VACOcast's patented *Vacuum Relief* insert. This creates a comfortable “perfect fit” with cast-like stability, to enable earlier weight bearing (when appropriate).

There is no inward pressure on the limb, which makes VACOcast notably comfortable—and extremely functional.


2012-03-13

**Why go Pro?** VACOcast Pro/Achilles is a functional orthotic boot with a unique hinge mechanism to adjust flexion and enable controlled Range Of Motion (ROM) for advanced physiotherapy—patients can be active while healing.

It is designed for a speedy recovery from fractures or Achilles repair.


## Progressive treatment options

### Achilles Ready

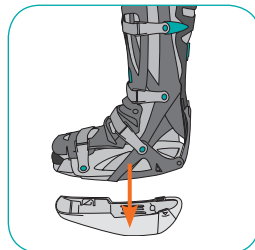


Removable fracture sole & Achilles wedge are both included.

### Precise Adjustments

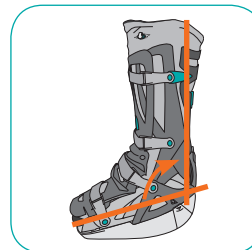

Gradually adjust flexion from  $-10^{\circ}$  to  $+30^{\circ}$ , with increments of  $5^{\circ}$ .

### Controlled ROM

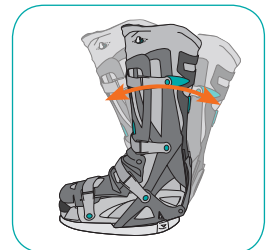

Keep in stationary position, or activate Range of Motion (ROM).

## Earlier weight bearing

The sooner patients can walk on their injured leg without crutches, the faster they are onto the road to recovery. The patented *Vacuum Relief* insert with micro-beads creates a comfortable “perfect fit” to enable earlier weight bearing (when appropriate).

## Incredibly comfortable

Patients rave about how comfortable their VACOcast Pro/Achilles is. Go to [www.vacocast.com/praise](http://www.vacocast.com/praise) to read what people are saying about their recovery with VACOcast.

